Air-Stirrup®

ENGLISH
BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:
The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE /INDICATIONS :
Provides support to prevent further injury by stabilizing the ankle. It may be suitable for use following acute injuries such as ankle sprains and Chronic Ankle Instability (CAI). Soft-goof/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS : N/A

WARNINGS AND PRECAUTIONS :
• If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately
• Do not use over open wounds.
• Do not use this device if it was damaged and/or package has been opened.
• If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION :

- Adjust heelpad width: Open brace and lift bottom of aircells until heel straps are fully exposed. Peel up heel straps and adjust heelpad width for a snug fit. Firmly press heel straps and aircells back in place. (Fig. 1)
- Apply brace: Place round edge of heelpad under heel. Align brace sides with ankle. Secure straps from bottom to top. Secure lower strap first for proper alignment. To ensure maximum comfort and support, wear an absorbent sock first and then a pair of lace-up athletic shoe or sport shoe. (Fig. 2)
- Adjust brace: Squeeze brace sides together with one hand. Tighten straps from bottom to top with other hand. Tighten for comfortable support. (Fig. 3)

ANKLE BRACE FIT ADJUSTMENT:

- Ankle Brace aircells are preinflated and normally do not require adjustment. If you experience pinching or uneven pressure:
 - Reposition brace or
 - Readjust heelpad and straps. Use lines on heelpad as a reference. If condition persists, adjust aircell volume (too much air will reduce support);
 - Unfold valve on top of aircell.
 - Insert inflation tube into valve 1/2 inches.
 - Remove a small amount of air by gently squeezing aircell or
 - Add a small amount of air by blowing into tube.
 - Squeeze valve flat just below tube, and slowly remove tube.
 - Fold valve.
- High Altitude : At high altitudes the aircells will expand beyond their optimal level and air may have to be removed from the aircells (see ankle brace fit adjustment). When flying, re-adjust straps to a comfortable pressure.

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LATEX FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2020 DJO, LLC
MD55 GmbH